



Advanced Chorion Membrane for Dental Procedures

DermaBind CH® is a natural chorion membrane, ethically sourced and processed to retain collagen-rich scaffolding proteins. Designed specifically for dental applications, it delivers a secure barrier, and lasting results for patients.

Key Clinical Advantages



Ready-to-Use

No trimming, pre-hydration, or orientation required.



Versatile

Compatible with implants, root surfaces, mesh, and palatal tissue.



Adaptive Fit

Conforms to complex wound surfaces, fully resorbs over time.



Convenient Storage

Sterile at room temperature, 5-year shelf life.

APPLICATIONS



Periodontal
pocket coverage



Gingival recession
treatment



Alveolar ridge
preservation



Implant site
protection

ABOUT THE PRODUCT

- **Composition:** Natural chorion tissue, rich in collagen (Types I, III, IV, V, VI).
- **Process:** Proprietary method preserving extracellular matrix, ensuring sterility (SAL 10⁻⁶) under FDA standards.

PREPARATION & PLACEMENT

1. Open sterile packaging using **aseptic technique**.
2. Place **directly** on the treatment site (periodontal pockets, gingival recession, ridge preservation).
3. Conforms **naturally**; fixation or suturing optional.
4. May be **layered** with other membranes or materials as needed.

SIZES & STORAGE

- Available in 3 sizes.
- Store at ambient temperature (do not freeze).
- 5-year shelf life.



30x30mm



20x20mm



10x10mm

FIXATION & DRESSING

1. Flexible enough to stay in place **without sutures**.
2. Can be secured with **clinician's preferred fixation method**.
3. Compatible with **secondary dressings or regenerative measures**.

MANUFACTURED BY



HealthTech
WOUND CARE

HealthTech Wound Care, Inc.
healthtechwoundcare.com
833-381-2402
© 2025

WMKT-5 v7

EXCLUSIVELY DISTRIBUTED BY

LEVARIS

Levaris, LLC
levarisglobal.com
954-477-8228

sales@levarisglobal.com
2950 N 28th Terrace
Hollywood, FL 33020



BILLING SUPPORT

reimbursement@healthtechwc.com