




DermaBind FM is a **dehydrated placental membrane covering** that preserves the comprehensive collagen matrix, glycoconjugates, and glycosaminoglycans.

THE DERMABIND FM PROCESS

DermaBind FM undergoes a proprietary preservation method, which retains all native layers of the placental membrane, including the amnion and chorion with the spongy layer intact, unlike many placental allografts on the market. The DermaBind FM process does not use unnecessary chemicals, antibiotics or preservatives and is never frozen.

The finished product is physician-friendly, durable, easily applied to a wound and can be stored at room temperature for up to 5 years.

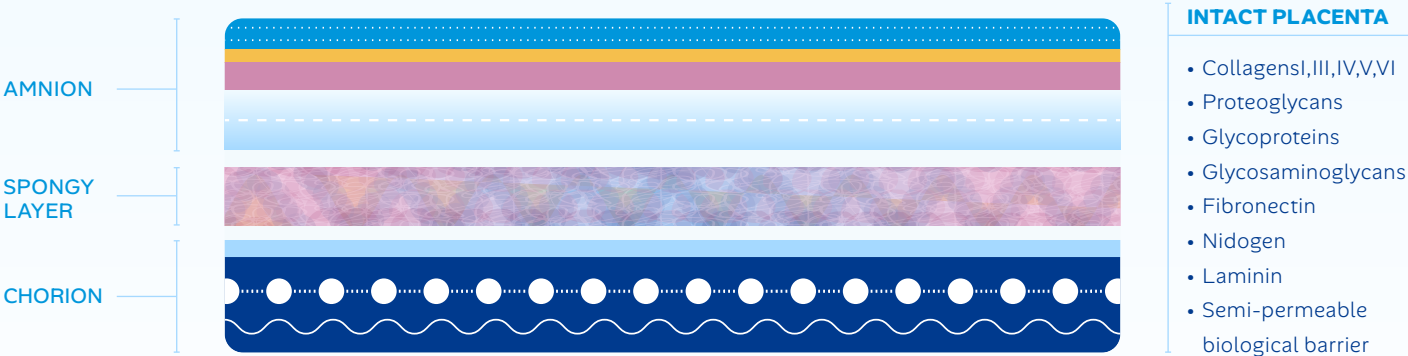

COMPOSITION

DermaBind FM contains 400+ proteins, including collagens, fibrins, elastins, glucosaminoglycans and glycoconjugates.

INTENDED USE


DermaBind FM can be used as a protective wound covering as either a partial and full thickness acute and chronic wounds.

- Trauma Wounds
- Diabetic Foot Ulcers
- Pressure Injuries (DFUs)
- Dehisced Wounds
- Venous Leg Ulcers (VLUs)

Can be used at the onset and for the duration of the wound, weekly or at the discretion of the treating physician.

SIZE & STORAGE

- Available in a wide range of shapes & sizes
- Maintain at ambient temperature prior to patient application
- Do not freeze
- May be stored for up to 5 years at room temperature.

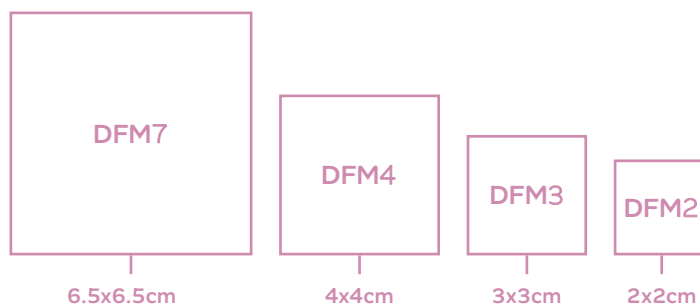

APPLICATION

1. Using **aseptic technique**, remove DermaBind FM from the package; the product may be trimmed as necessary.
2. Place the DermaBind FM patch **onto the wound**.
3. Assure DermaBind FM is in **intimate contact** with the wound bed; the product can be hydrated once placed on the wound bed.
4. Adhere DermaBind FM using the providers **preferred fixation method**, then cover DermaBind FM with a primary non adherent dressing of your choice (preferred method).
5. Apply **secondary dressings** specific to the wound type over the primary dressing of choice, which should remain in place for up to one week.

REIMBURSEMENT

- Medicare contractors cover DermaBind FM for VLUs and DFUs *
- DermaBind FM can be applied to, and billed for, other wound types
- Confirm coverage prior to treatment through our Benefits Program

DermaBind FM Billing

Q 4313

CPT CODES

Site Preparation CPT** 15002-15005

Application CPT codes** 15271-15278

* Consult with your LCD (local coverage determination) for the specific wound types covered. VLU = venous leg ulcer; DFU = diabetic foot ulcer.

**Subject to guidelines and approvals, please contact our billing team for further details.

MANUFACTURED BY



HealthTech Wound Care, Inc.
healthtechwoundcare.com
833-381-2402
© 2026

EXCLUSIVELY DISTRIBUTED BY

LEVARIS

Levaris, LLC
levarisglobal.com
954-477-8228

sales@levarisglobal.com
2950 N 28th Terrace
Hollywood, FL 33020



BILLING SUPPORT

reimbursement@healthtechwc.com