




DermaBind TL is a **dehydrated placental membrane covering** that preserves the comprehensive collagen matrix, glycoconjugates, and glycosaminoglycans.

#### THE DERMABIND TL PROCESS

DermaBind TL undergoes a proprietary preservation method, which retains all native layers of the placental membrane, including the amnion and chorion with the spongy layer intact, unlike many placental allografts on the market. The DermaBind TL process does not use unnecessary chemicals, antibiotics or preservatives and is never frozen.

The finished product is physician-friendly, durable, easily applied to a wound and can be stored at room temperature for up to 5 years.

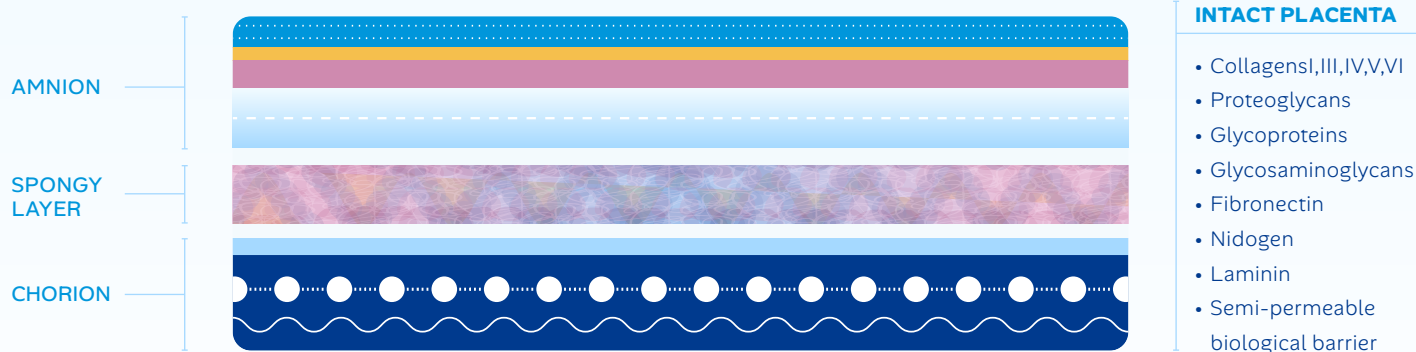

#### COMPOSITION

DermaBind TL contains 400+ proteins, including collagens, fibrins, elastins, glucosaminoglycans and glycoconjugates.

## INTENDED USE


DermaBind TL can be used as a protective wound covering as either a partial and full thickness acute and chronic wounds.

- Trauma Wounds
- Diabetic Foot Ulcers (DFUs)
- Pressure Injuries
- Venous Leg Ulcers (VLUs)
- Dehisced Wounds

Can be used at the onset and for the duration of the wound, weekly or at the discretion of the treating physician.

## SIZE & STORAGE

- Available in a wide range of sizes
- Maintain at ambient temperature prior to patient application
- Do not freeze
- May be stored for up to 5 years at room temperature.

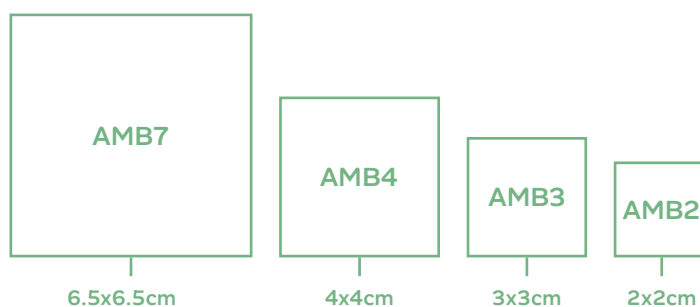

## APPLICATION

1. Using **aseptic technique**, remove DermaBind TL from the package; the product may be trimmed as necessary.
2. Place the DermaBind TL patch **onto the wound**.
3. Assure DermaBind TL is in **intimate contact** with the wound bed; the product can be hydrated once placed on the wound bed.
4. Adhere DermaBind TL using the providers **preferred fixation method**, then cover DermaBind TL with a primary non adherent dressing of your choice (preferred method).
5. Apply **secondary dressings** specific to the wound type over the primary dressing of choice, which should remain in place for up to one week.

## REIMBURSEMENT

- Medicare contractors cover DermaBind TL for VLUs and DFUs \*
- DermaBind TL can be applied to, and billed for, other wound types
- Confirm coverage prior to treatment through our Benefits Program

### DermaBind TL Billing

Q 4225

### CPT CODES

Site Preparation CPT\*\* **15002-15005**

Application CPT codes\*\* **15271-15278**

\* Consult with your LCD (local coverage determination) for the specific wound types covered. VLU = venous leg ulcer; DFU = diabetic foot ulcer.

\*\*Subject to guidelines and approvals, please contact our billing team for further details.

MANUFACTURED BY



HealthTech Wound Care, Inc.  
healthtechwoundcare.com  
833-381-2402  
© 2026

EXCLUSIVELY DISTRIBUTED BY

# LEVARIS

Levaris, LLC  
levarisglobal.com  
954-477-8228

sales@levarisglobal.com  
2950 N 28th Terrace  
Hollywood, FL 33020



BILLING SUPPORT

reimbursement@healthtechwc.com